

## neoform® Rapid – Rapid disinfectant for surfaces & medical devices



Product in eShop

[Home](#) / [Assortment](#) / [Medicine & Healthcare](#) / [Safety & Hygiene](#) / [Surface cleaning & disinfection](#) / neoform® Rapid – Rapid disinfectant for surfaces & medical devices

### Product Data

#### Rapid disinfectant for surfaces & medical devices

**Alcoholic, ready-to-use solution for targeted rapid disinfection of surfaces of medical devices, e.g. operating tables, in hospitals and medical and dental practices**

- bactericidal
- levurocide
- fungicide
- tuberculocide
- limited virucidal
- limited virucidal PLUS (effective against norovirus and adenovirus)
- effective against rota and corona viruses

**Use by wiping or spraying. VAH (Verbund für Angewandte Hygiene)-listed and registered in the virucidal list of the IHO (Industrieverband Hygiene und Oberflächenschutz).**

- UN 1987 Propan-1-ol, ethanol, solution, 3, III, D/E
- Incl. spray head

#### Special features

- Comprehensive effectiveness already after a short exposure time
- Fast and residue-free drying, no rinsing necessary

- Free from aldehydes, dyes and perfume

### Application

Apply neoform Rapid undiluted to the cleaned, dry surfaces and wipe the surfaces to be disinfected. The surfaces must be completely wetted with neoform Rapid.

### Area of application

- Targeted rapid disinfection of
- Surfaces of medical devices, such as operating tables, as well as other surfaces in hospitals and medical and dental practices
- Surfaces and equipment in the pharmaceutical and cosmetics industry
- surfaces in the laboratory area, e.g. laboratory benches

### Range of services

- Bactericidal, levurocidal, fungicidal, tuberculocidal efficacy according to VAH1 and EN methods<sup>2</sup>, as well as limited virucidal efficacy according to RKI/DVV3 methods confirmed by expert opinion
- Furthermore, virucid PLUS (effective against noroviruses and adenoviruses) and effective against rotaviruses
- The surfaces to be disinfected must be resistant to alcoholic disinfectants
- Not suitable for disinfecting acrylic glass (Plexiglas)
- VAH1-listed
- Added to the IHO5 disinfectant list
- The application in the Dr. Weigert fleece cloth dispenser neoform wipes RTF meets the requirements of the current VAH recommendation



Item No.	execution	capacity	Price
18.9445.00	bottle incl. spray head	0,75 liter	CHF 18.35 <del>CHF 31.30</del>
18.9445.01	bottle incl. spray head	12 x 0,75 liter	CHF 251.05

\* The prices are non-binding and are to be understood as selling prices in Swiss francs without value added tax (VAT), as well as all other fees, charges and taxes. The prices displayed in the eShop may differ from the PDF file due to regular updates.

\*\* Please note that when ordering chemicals and detergents, transport and packaging costs for hazardous goods as well as legally prescribed fees are charged. These will be shown in detail on the order confirmation, which you will receive in addition to the confirmation of receipt.

\*\*\* Further information such as technical information and safety data sheets can be found online in our eShop.

\*\*\*\* The PDF file was created on [www.huberlab.ch](http://www.huberlab.ch) on 01.05.2024 at 04:01 o'clock.